GB

Instructions for use of disposable obstetrical pessary

Disposable obstetrical pessary (further obstetrical pessary) is designed for the prophylaxis of preterm delivery among pregnant women with isthmic-cervical insufficiency (ICI) and for the prophylaxis of development of this pathology.

Indications for use

1. Functional and organic isthmic-cervical insufficiency.
2. Prophylaxis of isthmic-cervical insufficiency among pregnant women.
3. Prophylaxis of suture dehiscence in case of surgical correction of ICI.

Description of obstetrical pessary

Obstetrical pessary (fig. 1) is manufactured from the biologically inert material in form of trapezium with smoothed semi-ring 1 corners. Large 2 and small 3 bases are made concave. The surfaces of the bases prevent the compression of straight intestine (large base) and urinary bladder (small base). The central hole 4 designed for cervix uteri is situated closer to the wide base. On the periphery from the central hole are situated holes 5 for vaginal secretion drainage.

Mechanism of action

The mechanism of action of obstetrical pessary is based on the decrease of pressure on cervix uteri owing to the decrease of pressure of ovum.

Isthmic-cervical insufficiency (fig. 2): is open outer a and inner b mouth, fetal membranes c prolapse into the cervical canal.

Obstetrical pessary is inserted (fig. 6): cervix uteri in the central hole 4 of pessary, large base 2 is placed in the posterior vaginal vault, small 3 – in the anterior vaginal vault. The lateral surface of pessary touches the walls of the vagina which embrace and hold it. The pressure on cervix uteri is reduced.

Contraindications

- Absolute:**
- the present method should not be used in clinical situations when the prolongation of pregnancy is unreasonable;
 - anatomic features of a patient which prevent the correct placement of pessary.
- Relative:**
- inflammatory diseases of vagina, cervix uteri, external genital organs (preliminary sanitation with the following bacteriological control is necessary).

Warnings

- During the use of obstetrical pessary the increase of vaginal secretion is possible;
- obstetrical pessary is not designed for the correction of genitals prolapse;
- it is unacceptable to use the pessary in case of damaged final pack or after the expiry date indicated on the pack.

Choosing the type of obstetrical pessary

Choosing the type of obstetrical pessary depends on the individual anatomic features of a patient. The recommendations on choosing the type of obstetrical pessary are given in table 2.

Insertion and removal of obstetrical pessary

Insertion and removal of obstetrical pessary is a medical manipulation. The procedure may be carried out in ambulatory and hospital conditions. Insertion and removal of pessary do not require anaesthesia.

In order to ease the insertion it is expedient to treat the pessary and genitals of a patient with the ointment allowed for use for pregnant women. Conditions for insertion of pessary:

- absence of contraindications for use;
- normal uterine tone;
- consent of a patient.

The insertion of pessary is carried out with the observation of general rules of aseptics.

After the examination of a woman with an empty urinary bladder place the obstetrical pessary in such a way, that the wide base is placed near the opening of the vagina. Insert the lower semi-ring of the wide base into the vagina (fig. 2).

Then by pushing the posterior wall of the vagina insert the upper semi-ring of the wide base. After that fully insert the obstetrical pessary (fig. 3).

Rotate the obstetrical pessary so that the wide base is placed in the posterior vaginal vault and cervix uteri – in the central hole of the obstetrical pessary (fig. 4, 5, 6).

Removal of obstetrical pessary

Pessary is routinely removed at 37-38 weeks of pregnancy both in ambulatory and hospital conditions. The method of removal is inverse to Cthe insertion. After the extraction of the obstetrical pessary it is expedient to carry out the sanitation of genital tracts depending on the character of vaginal microflora.

A number of clinical situations require preterm removal of pessary:

1. necessity of emergency delivery;
2. discharge of amniotic fluid;
3. development of labor activity;
4. chorioamnionitis;
5. appearance of blood-tinged discharge from genital tracts (in case of necessity repeated insertion of pessary is possible);
6. pain syndrome stipulated by the obstetrical pessary (the insertion of pessary of smaller size is possible).

Examination and monitoring of pregnant women who are using the pessary

Bacterioscopic examination of vaginal smears every 14-21 days is obligatory. The control of cervix uteri state must be carried out with the orientation at the data of ultrasonic examination in the dynamics (every 3-4 weeks). The treatment of ICI with the help of obstetrical pessary may be combined with any drug therapy and with the surgical method of correction of isthmic-cervical insufficiency. During the use of the obstetrical pessary it is expedient to recommend patients sexual rest. In case of development of colpitis sanitation is possible with the presence of the obstetrical pessary, in case of absence of effect from sanitation with the presence of the pessary it is expedient to remove the device, to carry out the sanitation, then to insert the pessary.

Recommendations for patients

Obstetrical pessary is designed for the prophylaxis of preterm delivery in case of cervix uteri incompetence (isthmic-cervical insufficiency). In case of discovery of this pathology the device is inserted into the vagina in order to reduce the pressure on the "incompetent" cervix uteri. The terms of stay of pessary in the vagina are determined individually. Pessary is routinely removed at 37-38 weeks of pregnancy. A number of clinical situations require preterm (temporary or final) removal of pessary.

During the use of the obstetrical pessary the regular surveillance of the obstetrician-gynecologist is necessary, it is expedient to observe sexual rest, the increase of vaginal discharge from the genital tracts is possible (it is necessary to exclude inflammation).

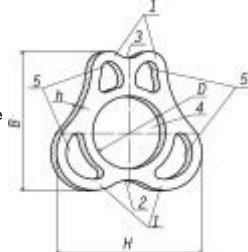
Fig. 1 General view of obstetrical pessary

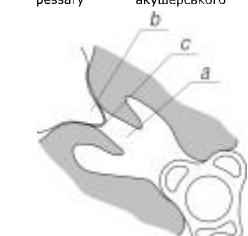
Fig. 2 Isthmic-cervical insufficiency

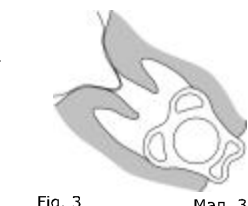
Fig. 3 Insertion of the pessary

Fig. 4 Pushing the posterior wall of the vagina

Fig. 5 Rotating the pessary

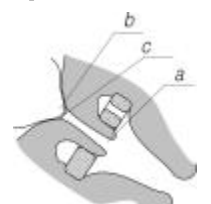
Fig. 6 Insertion of the pessary

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